


Patriot LX Bed Mount Trapeze Clamp Location

When using bed mount (clamp on) style trapeze with the Patriot LX Models US5000 and US6000 follow the assembly and safety instructions provided with your trapeze. Positioning of the trapeze clamps on the Patriot LX is shown in the pictures below.

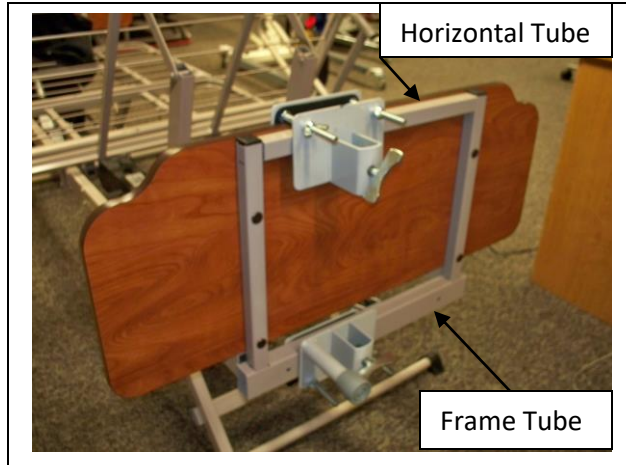

The Top Clamp is mounted to the horizontal support tube and headboard. Do not attach solely to the headboard. The horizontal support tube must be assembled on the headboard.

The Bottom Clamp is mounted directly to the bed frame end tube, not the headboard.

The Top Clamp and Bottom Clamp are located to either side of the hi-low screw rod on the US5000 Semi Electric and adjacent to the head hi-lo motor on the US6000 Full Electric pictured.



The headboard horizontal and vertical mounting tubes are standard with all Patriot LX Beds. The Patriot LX Bed can be lowered to a deck height of 10 inches. Place the trapeze vertical mast high enough so as not to hit the floor when in the low position.