

Operation Manual

Rhino P260L



Bariatric Trapeze Bar

Weight Limited - 500Lbs

TABLE OF CONTENTS

Forward	2
Safety Precautions.....	2
P260L Layout.....	3
Assembly Instructions	4 - 7
Note.....	8
Maintenance	9
Warranty Information	10

Limited Warranty

The warranty below has been drafted to comply with Federal Law applicable to products manufactured after July 4, 1975. This warranty is extended only to the original purchaser/consumer (or dealer non-consumer who does not buy for resale).

Tuffcare® warrants that its proprietary Rhino Series patient lifters will be free from defective workmanship and materials for a period of one (1) years following the date of original purchasing date with the following exceptions: All welds are warranted for the life of the patient lifter. Electronics on the lifter are warranted for one (1) year. The warranty period commences on the original manufacture date. If within such warranty period any such product proven to Tuffcare® satisfaction to be defective, such product will be repaired or replaced at Tuffcare® option.

Tuffcare® sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement. This warranty does not include any labor charges incurred in replacement parts installation. Freight charges to the factory are at the expense of the consumer or seller. Return freight charges will be prepaid by Tuffcare®. For warranty service, please contact the authorized dealer from whom you purchased your Tuffcare® products. In the event that you do not receive satisfactory warranty service, please write directly to Tuffcare®. DO NOT RETURN PRODUCTS WITHOUT PRIOR AUTHORIZATION.

LIMITATIONS AND EXCLUSIONS: The foregoing warranty shall not apply to products subjected to negligence, abuse, misuse, improper operation, improper maintenance, improper storage or damages beyond Tuffcare® control. The evaluation will be solely determined by Tuffcare®. The warranty shall not apply to problems arising from normal wear, or failure to follow instructions, or if parts are not manufactured by Tuffcare®, or not comply with original equipment specifications that are added to Tuffcare® product.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. TUFFCARE® SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

This warranty gives you specific rights and you may also have other legal rights which vary from state to state. Some states do not allow the exclusion or limitations of incidental or consequential damage, or limitation on how long an implied warranty lasts, therefore, the above exclusion and limitation may not apply to you.

MAINTENANCE

1. Clean patient trapeze bar regularly with non abrasive cleaners.
2. Check monthly that all screws are tight and have not loosened.
3. Make sure all rubber tips and ends are in good condition for use.

FORWARD

Congratulations on your purchase of a TUFFCARE Rhino series patient trapeze bar! Our Rhino trapeze bars are engineered and designed with durability and easy maintenance in mind.

Rhino trapeze bar will allow patients to lift themselves while in bed.


Please read through this manual and get acquainted with your patient trapeze bar. Follow the maintenance schedule to ensure proper function of your unit.

SAFETY PRECAUTIONS

- * Do not operate your patient trapeze bar unit until you read and fully understand this manual.
- * Do not operate your Rhino trapeze bar until it is fully assembled and checked.
- * Do not exceed the maximum weight capacity of the trapeze bar.

Do not modify or replace parts with other than those manufactured by TUFFCARE.

RHINO P260L LAYOUT

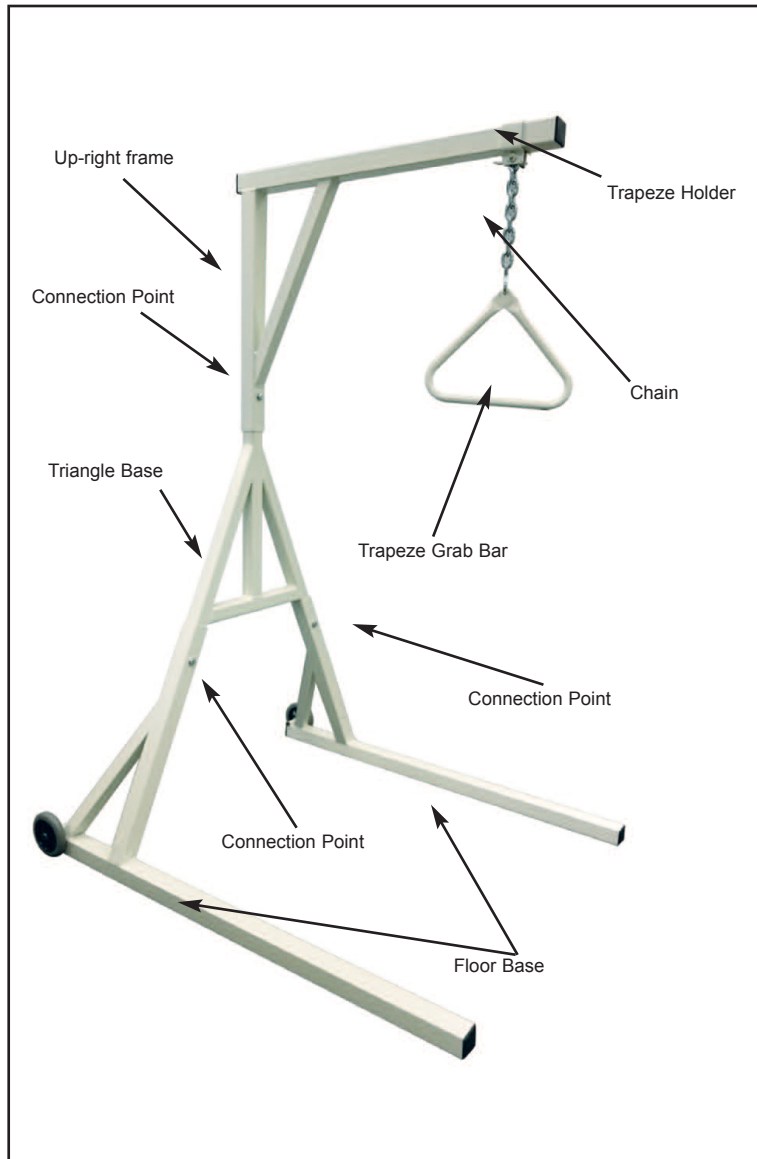

Note:


ASSEMBLY INSTRUCTIONS

7. Attach the Trapeze Grab Bar to the Trapeze Holder, then tighten it. Check all bolts to make sure that they are all tightened.



ASSEMBLY INSTRUCTIONS

- A. Please carefully unpack the container and remove the patient Lifter from the shipping carton.
- B. Inspect the packaging, make sure there's no shipping damage and all the components are in the shipping carton.
 2. Floor Bases.
 - 1 Triangle Frame.
 - 1 Up-right Frame.
 - 1 Trapeze Grab Bar (with chain)
 - 3 Completed Screw-washer-nut-bolt sets.

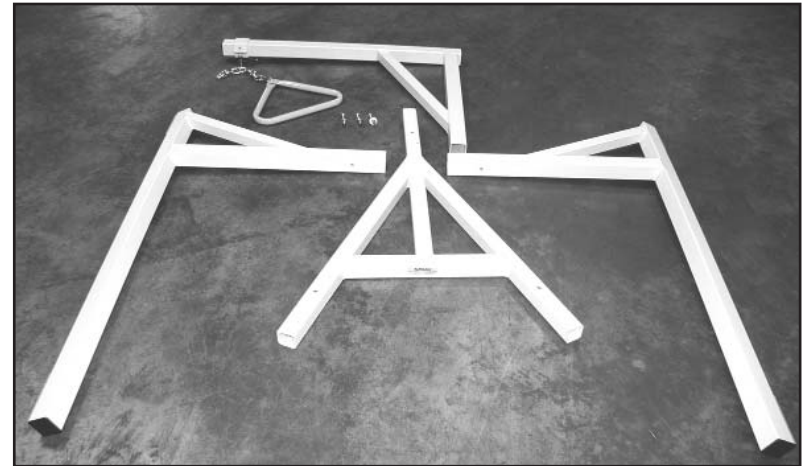

- C. Assistance is needed and highly recommended due to the heavy weight of parts in completing the final assembly.

ASSEMBLY INSTRUCTIONS

1. Insert Triangle Frame to the Floor Base.
2. Assistance is needed and highly recommended due to the heavy weight of parts in completing the final assembly.

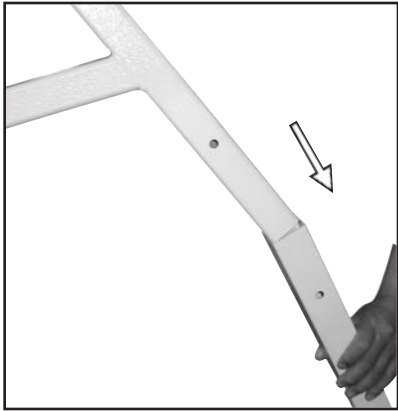


Figure 1



Figure 2

3. Confirm height and holder settings if they are appropriate, then tighten them. Please, check all bolts to make sure that they are all tightened
4. Repeat step 1 to 3 for the other floor base.

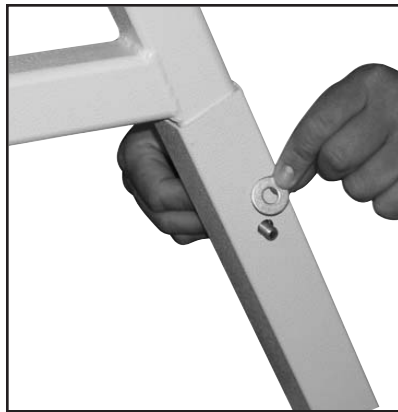


Figure 3



Figure 4

ASSEMBLY INSTRUCTIONS

5. Insert Up-right Frame to the top of the Triangle Base.

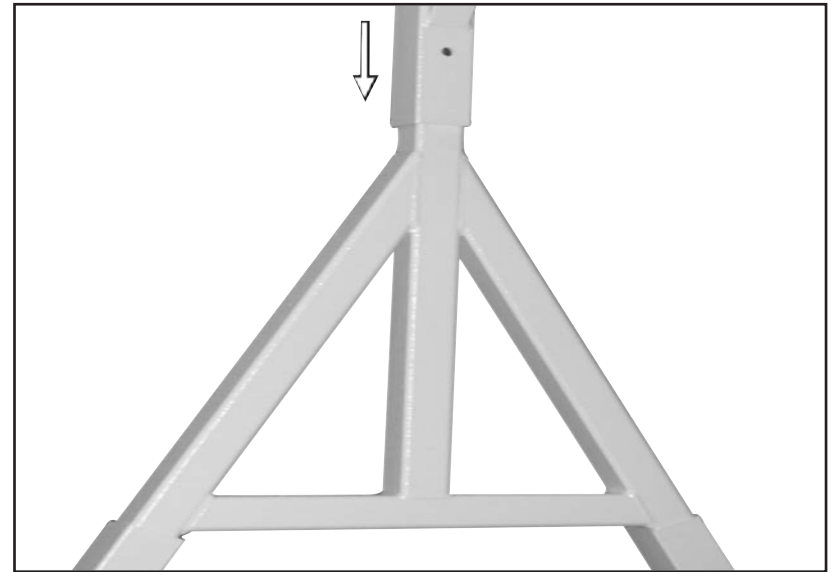


Figure 5

6. Again, confirm height and holder settings if they are appropriate, then tighten them. Check all bolts to make sure that they are all tightened.

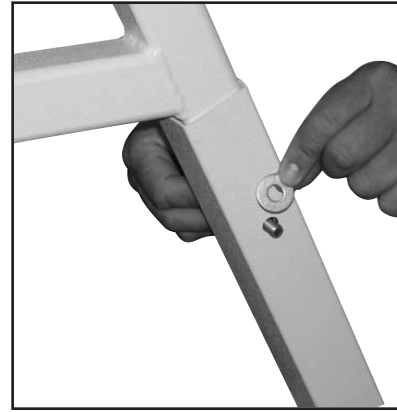


Figure 6



Figure 7